Dr.Lensor CL SOLUTION

MULTIPURPOSE WITH SODIUM HYALURONATE
FOR THE DISINFECTION, PRESERVATION, CLEANING, RINSE,
HYDRATION OF ALL TYPES OF SOFT CONTACT LENSES
SUITABLE FOR SENSITIVE EYES, ACTIVE IN 5-7 MINUTES

PROPERTIES: *Dr.Lensor CL SOLUTION* has a high "stabilizer effect" towards the interfaces of contact lenses/tear film/epithelium and "normalizing" action of the tear film thanks to the unique excellent muco-adhesive and bio-adhesive properties of HA and Tamarind.

STERILE | A

INDICATIONS: It's suitable for preservation, disinfection, rinse, cleaning, hydration, daily protein removal of all kinds of soft contact lenses.

INSTRUCTIONS FOR USE:**Method 5-7 minutes**

-Wash hands and dry accurately with a clean cloth. - To avoid mistakes, start the treatment always with the same lens. - Remove the lens from the eye and put it on the hand palm adding a quantity of *Dr.Lensor CL SOLUTION*, enough to wet the lens completely. - Rub softly with "backwards and forwards" movements, both lens surfaces for 10-15 seconds, using a finger of the other hand. -Rinse carefully and abundantly the lens with *Dr.Lensor CL SOLUTION* in order to remove all deposits. - Put the lens in the container provided. - Repeat the same operations with the other lens. - Fill the container with *Dr.Lensor CL SOLUTION* until the dosage line. - Close the container accurately and leave the lenses immersed for at least 5 – 7 minutes. - Before wearing the lenses, rinse them with *Dr.Lensor CL SOLUTION*.

"DON'T TOUCH" Method

Follow "Method 5-7 minutes" preliminary instructions but leave the lenses immersed in the provided container for 2-3 hours or all night without rubbing them. Rinse the lenses with *Dr.Lensor CL SOLUTION* before wearing them.

CONTRAINDICATIONS: There are no ascertained contraindications, apart from allergy or specific hypersensitivity to some component.

PRECAUTIONS: *Dr.Lensor CL SOLUTION* once open, must be used within three months. The left liquid must be disposed. Do not use in case of damaged seal, as sterility cannot be guaranteed. To avoid any contamination, do not put the dropper in contact with any kind of surface. Ensure that the bottle is well closed after product use. Store the bottle away from heat founts and in any case under 30°C. Do not use after the expiry date. 

WARNINGS: For lenses remained in the *Dr.Lensor CL SOLUTION solution* over 3-4 weeks, before use eliminate the old solution from the container and proceed with disinfection, at least according method 1 (5 – 7 minutes). Do not use in case of specific hypersensitivity to some component. Keep it far from children reach. Use always fresh solution getting rid of the one remained in the lens-keeping container. Do not replace with a different disinfectant. Seek professional advice before using any ophthalmic drug while wearing contact lenses. Never use tap water for soft contact lenses.

COMPLICATIONS: In case of persistent trouble or irritation, discontinue use and consult a specialist.

VALIDITY: 3 years if intact and properly preserved.

 Follow the instructions on the label strictly and keep this technical-informative note.

COMPOSITION: Polyhexamethylene Biguanide 0.0001%, Disodium edetate 0.1%, Sodium Hyaluronate 0.01%, Tamarind 0.005%, made up to 100% with Excipients and Purified water.

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